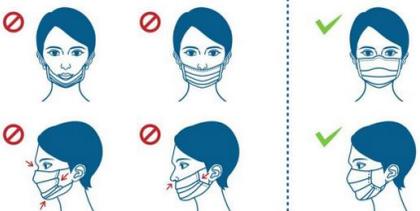
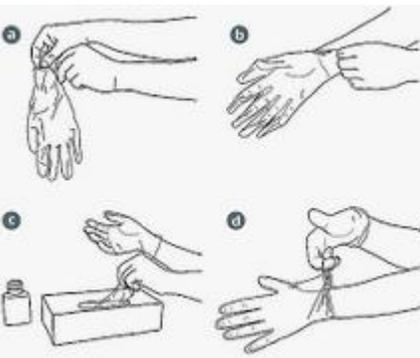
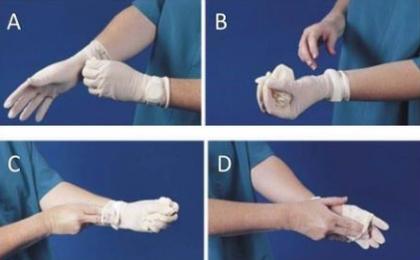


DONNING (Putting on)	<p>Disposable Gowns</p> <ul style="list-style-type: none"> Fully cover torso from neck to knees, arms to end of wrists, and wrap around the back Fasten in back of neck and waist <p>Reusable Lab Coats</p> <ul style="list-style-type: none"> Insert one arm to a sleeve at a time Adjust the shoulders and sleeves until comfortable Close the front of the coat by buttoning all the buttons and make sure that the coat is knee length 	<p>Face Coverings/Masks</p> <ul style="list-style-type: none"> Wear the clean cloth face covering by securing it with ties, ear loops, or tying behind the head Make sure it fits snugly but comfortably against side of the face (make sure hands are clean if any adjustments are needed) 	<p>Goggles or Reusable Face Shields</p> <ul style="list-style-type: none"> Place over face and eyes and adjust to fit Face coverings are required to be worn underneath the face shields 	<p>Disposable Gloves</p> <ul style="list-style-type: none"> Check for holes Put on gloves Move hands around 
DOFFING (Taking off)	<p>Disposable Gowns</p> <ul style="list-style-type: none"> Unfasten gown ties, taking care that sleeves don't contact your body when reaching for ties Pull gown away from neck and shoulders, touching inside of gown only Turn gown inside out Fold or roll into a bundle and discard in a regular waste container <p>Reusable Lab Coats</p> <ul style="list-style-type: none"> Release closer and open front of coat Pull the sleeve and slide fingers under cuff Pull arm completely from sleeve Repeat pulling the sleeve, sliding fingers off under and pulling arm completely from the sleeve on the other side  <p>Cleaning cloth face coverings</p> <ul style="list-style-type: none"> Laundry face coverings daily Dry in high heat  <p>Cleaning goggles and face shields</p> <ul style="list-style-type: none"> Disinfect hands before cleaning Apply disinfectant to paper towel or use alcohol cleaning wipes Wipe the front and back of the face shield and goggles Wipe the straps Wipe the foam Place goggles or face shields in an open bag to dry and keep clean until next use 	<p>Disposable Gloves</p> <ul style="list-style-type: none"> Pinch glove and roll it down until the glove is completely off in a ball in the palm of the other hand Then slide a finger(s) down and inside the glove on the other hand and pull it off until it's balled around the first glove Throw gloves away immediately and wash hands with soap and water for 20 seconds 	<p>Disposable Gloves</p> <ul style="list-style-type: none"> Pinch glove and roll it down until the glove is completely off in a ball in the palm of the other hand Then slide a finger(s) down and inside the glove on the other hand and pull it off until it's balled around the first glove Throw gloves away immediately and wash hands with soap and water for 20 seconds 	

**Lab coats are laundered by the university, do not dispose of them. Place used coats into provided bins.